


1.


### Assembling the Panel Mount Option

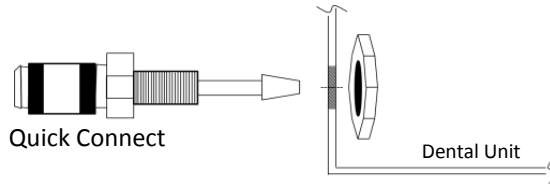

Drill a 7 mm hole into dental unit, remove nut, insert quick connect through hole, fasten nut onto quick connect with 11 mm wrench.

OR


### Assembling the Hanger Mount Option

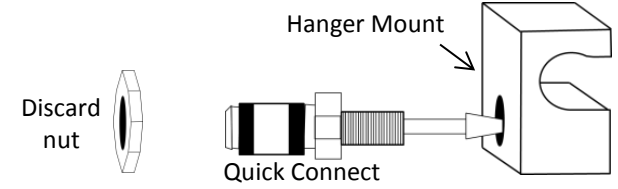

Remove the nut from the quick connect and discard. Thread quick connect into the hanger mount with 12 mm wrench.

2.

### Install the Toggle Switch\* to the Foot Pedal

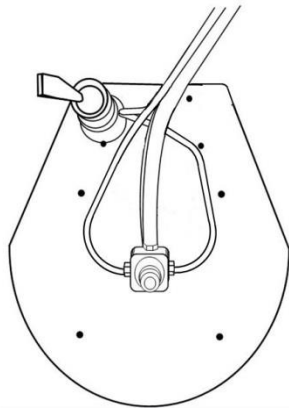


Utilize constant air flow from the foot pedal manifold to supply the toggle switch.

*\*Install the toggle switch (not included) with the appropriate toggle switch that the foot pedal manufacturer recommends.*

### Install the Swing Arm Option


Underneath the dental unit, locate the two available machined holes opposite the tray mount.

Install the AquaSept™ Swing Arm with the appropriate sized screws supplied in installation kit.

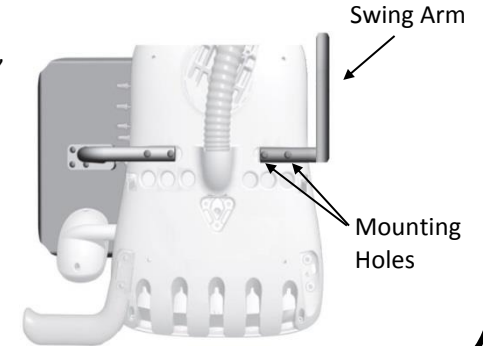


3.

### Surface Mount Option

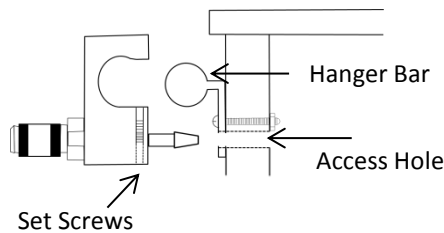


Mark where the barb exits the hanger mount. Drill a 9 mm access hole on this mark. Thread hose through access hole and fasten hose to barb. Install the hanger mount and tighten set screws of hanger mount onto hanger bar.

### Standard Mount Option

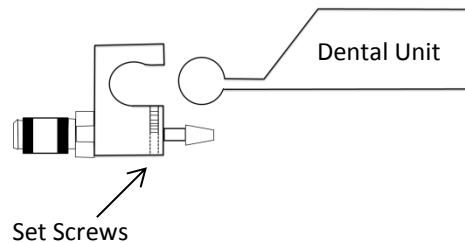

Tighten set screws of hanger mount onto dental unit 1/2" bar.

### Swing Arm Mount Option

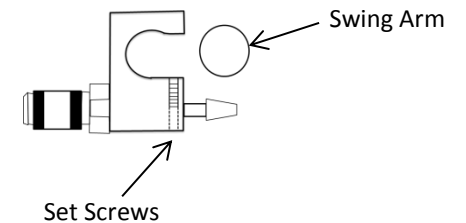


Tighten set screws of hanger mount onto swing arm 1/2" bar with allen wrench.



## INSTALLATION FOR ELECTRIC HANDPIECES


Assemble and mount the quick connects first,  
then install the toggle switch onto foot pedal.  
See back of page for instructions.

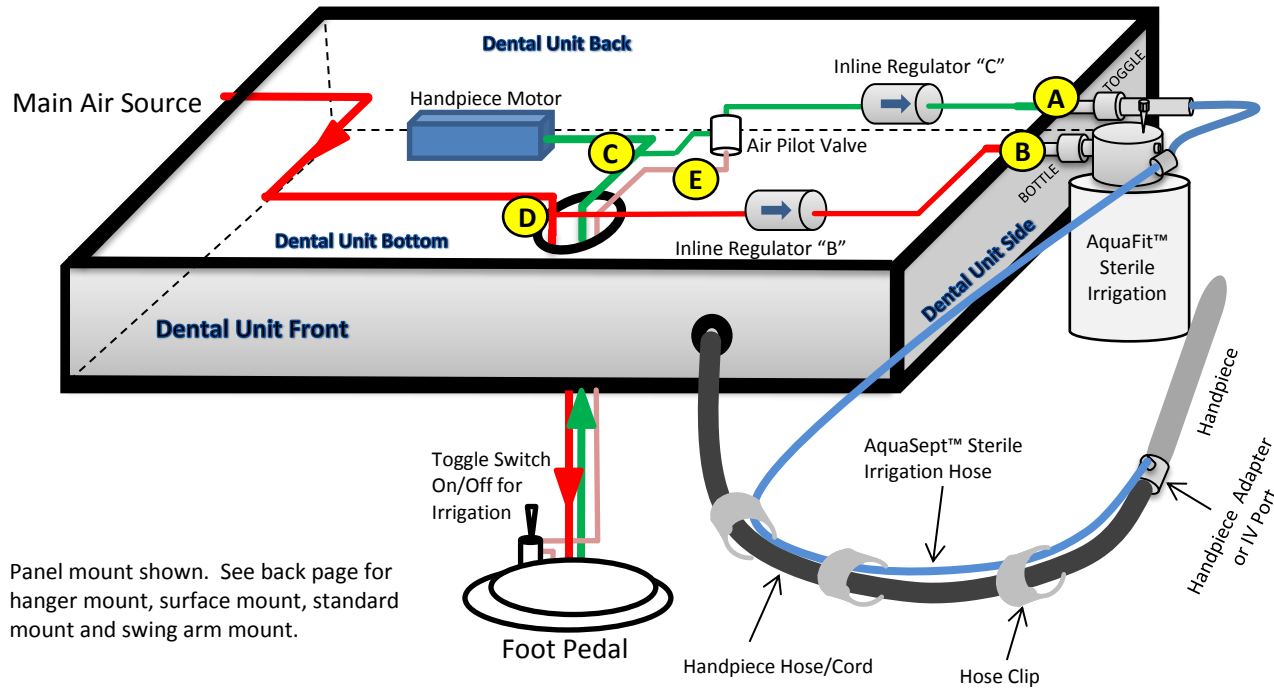


Panel mount shown. See back page for  
hanger mount, surface mount, standard  
mount and swing arm mount.

- Label quick connect "A" with the "Toggle" label provided. Connect the hose labeled "A" onto the "Toggle" quick connect. Connect other end of hose to regulator "C".
- Label quick connect "B" with the "Bottle" label provided. Connect the hose labeled "B" onto the "Bottle" quick connect. Connect other end of hose to regulator "B".
- Splice (Tee) the foot pedal return air line with the hose labeled "C".
- Splice (Tee) the main air line with the hose labeled "D".
- Splice (Tee) hose "E" to the toggle switch from the foot pedal.


### A & B

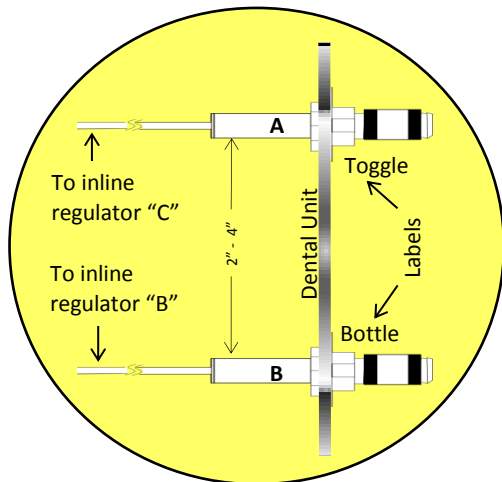


### C

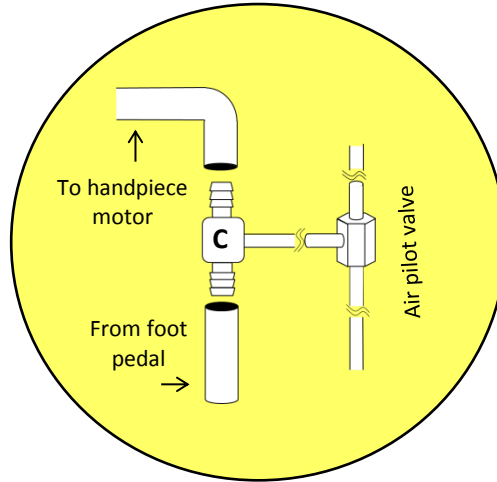

### D

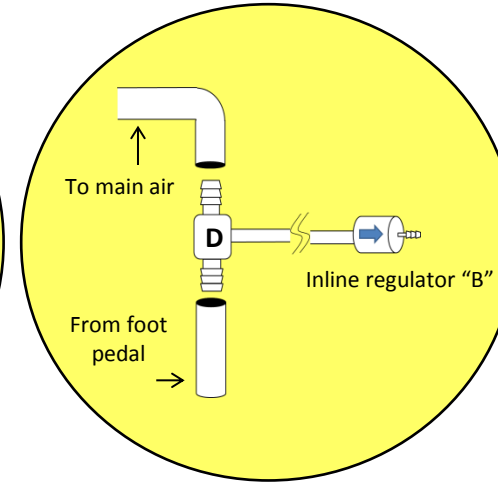

### E

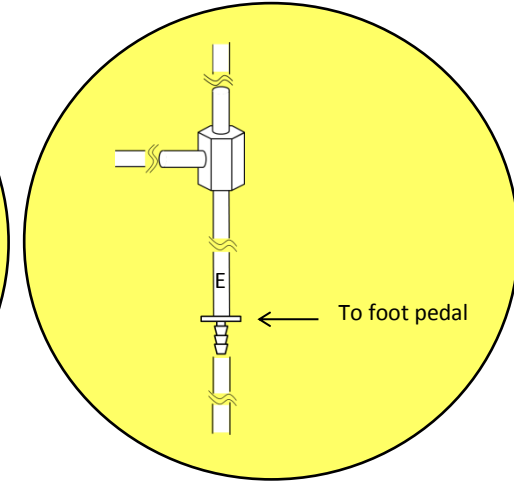

[www.aquasept.com](http://www.aquasept.com), [info@aquasept.com](mailto:info@aquasept.com)

Toll Free: 888-539-3907, Local: 715-639-2435, Fax: 715-639-9205