


1.


Assembling the Panel Mount Option

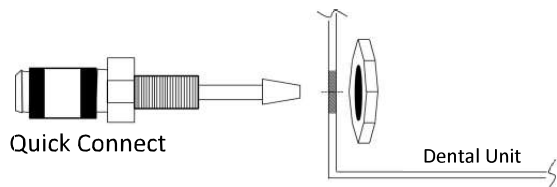

Drill a 7 mm hole into dental unit, remove nut, insert quick connect through hole, fasten nut onto quick connect with 11 mm wrench.

OR


Assembling the Hanger Mount Option

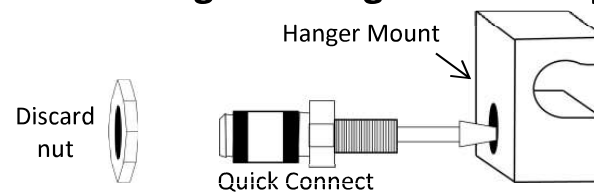

Remove the nut from the quick connect and discard. Thread quick connect into the hanger mount with 12 mm wrench.

2.

Install the Toggle Switch* Option

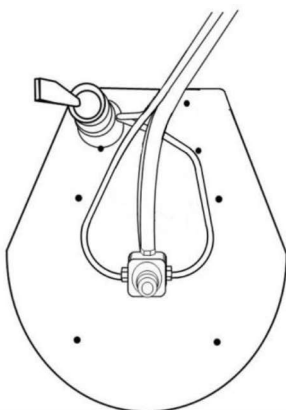


If the existing foot pedal does not have a toggle switch, install a toggle switch as shown. Utilize constant air flow from the foot pedal manifold to supply the toggle switch. If the foot pedal already has a toggle switch, skip this step.

**Toggle switch not included. Contact the foot pedal manufacturer for selecting appropriate foot pedal toggle switch.*

Install the Swing Arm Option


Underneath the dental unit, locate the two available machined holes opposite the tray mount.

Install the AquaSept™ Swing Arm with the appropriate sized screws supplied in installation kit.

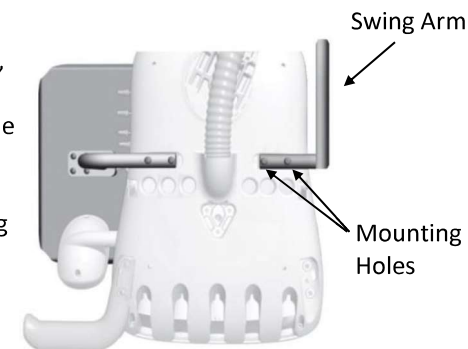


3.

Surface Mount Option

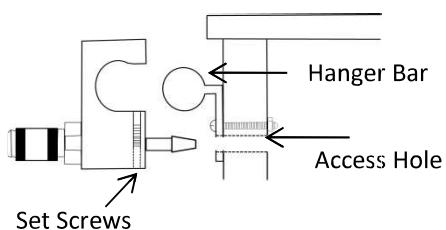


Mark where the barb exits the hanger mount. Drill a 9 mm access hole on this mark. Thread hose through access hole and fasten hose to barb. Install the hanger mount and tighten set screws of hanger mount onto hanger bar.

Standard Mount Option

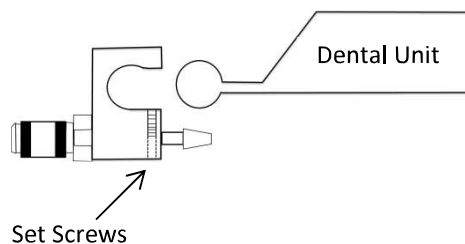

Tighten set screws of hanger mount onto dental unit 1/2" bar.



Swing Arm Mount Option

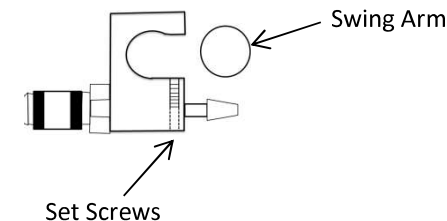

Tighten set screws of hanger mount onto swing arm 1/2" bar with allen wrench.

Bypassing the foot pedal toggle option for AquaFit™


Instructions for splicing off of the Dental Unit drive air rather than utilizing the foot pedal for On/Off function.
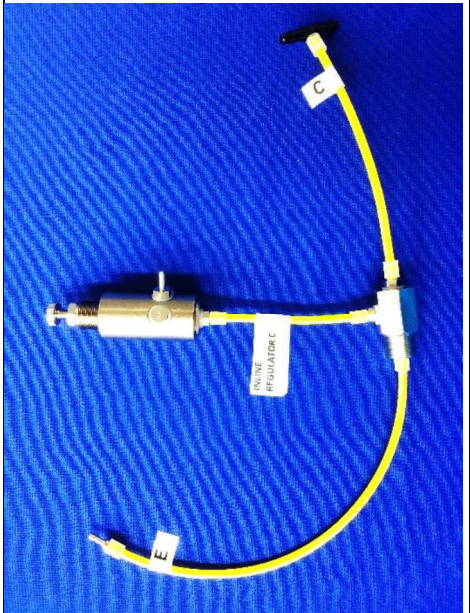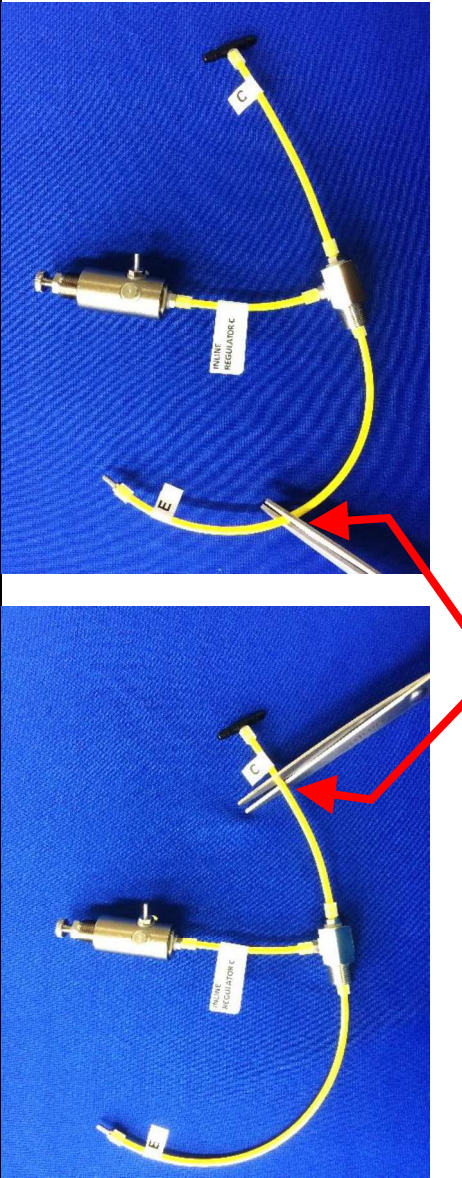
<p>STEP 1. Gather the “E” and “C” assembly from the 10228 AquaFit™ Installation Kit.</p>	
<p>STEP 2. Cut the “E” and “C” ends off their respective hoses.</p>	


Cut approximately 1” from “C” label and 3” from “E” label

Bypassing the foot pedal toggle option for AquaFit™

Instructions for splicing off the Dental Unit drive air rather than utilizing the foot pedal for On/Off function.

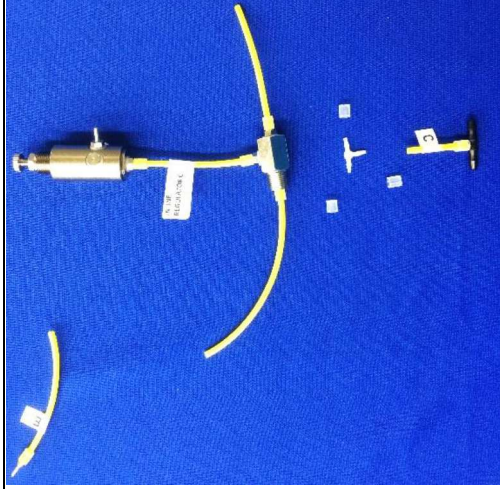

Gather supplied collars and plastic tee.

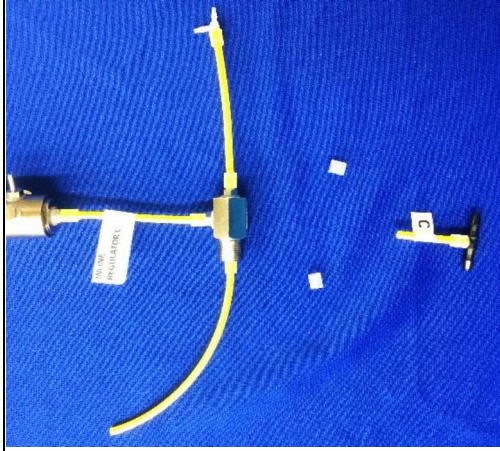

Slide collar over hose, press hose onto splice and slide collar over hose.




Repeat process.



STEP 3.

Assemble the collars over all three hoses for joining the tee as shown in the pictures.

NOTE: Press hoses over barbs of tee and then slide the collars over the hoses.

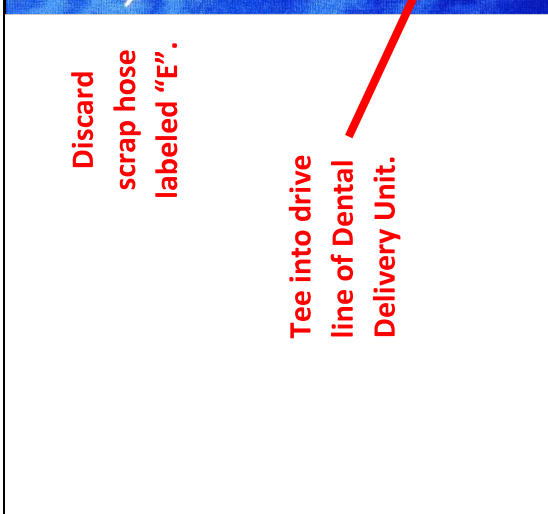

Discard scrap hose labeled "E".

Tee into drive line of Dental Delivery Unit.

STEP 4.

Finished assembly.

Splice the "C" hose into the drive line and proceed with installation instructions (disregarding Step E).

Discard the scrap hose labeled "E".