

AquaSept™/AquaFit™ Limited Warranty Policy

Scope of Warranty

Genesis Dental Technologies, LLC, manufacturer of AquaSept™/AquaFit™ (“Genesis”) warrants to the original retail purchaser that the AquaSept™/AquaFit™ branded products sold to the original purchaser will be free from defects in material and workmanship under normal use and service for a period of one year from the date of invoice. Warranty includes parts and labor. Shipping costs are excluded. This limited warranty shall only apply to defects that are reported to Genesis within the warranty period and which, after examination by Genesis, prove to be defective. If Genesis finds the product to be defective during the applicable warranty period, Genesis shall correct any such defect by either (a) repairing, adjusting, or replacing any defective or non-conforming product, or defective or damaged part or parts of the product or (b) by making available at the place of installation any necessary repaired or replacement parts, at Genesis’ option. This warranty is not transferable or assignable and only extends to the first retail purchaser.

Accessories and parts not supplied by AquaSept™ are excluded from this limited warranty, including but not limited to sterile pour bottles sourced from other suppliers. Sterile Pour Bottle Adaptors previously supplied are excluded from this warranty.

Genesis is not obliged to carry out dismantling or re-installation of any AquaSept™/AquaFit™ part. Any defects in a repaired or replaced part shall be covered to the extent of the unexpired term of the original warranty period. Warranty does not cover consumable parts, or normal wear parts, or any goods that have been supplied by any company other than Genesis.

Exclusions-Limitation of Liability

Genesis shall be liable under this limited warranty only for defects that appear under normal conditions of operation and proper use. Genesis shall not take any responsibility whatsoever for the product if stored or moved carelessly; tampered with, altered or misused; customer fails to seek and obtain repair or replacement in a timely manner; or product is excessively used. In addition, if the Product has been installed, serviced or repaired by anyone other than an authorized service provider or Genesis, any defects caused or contributed to the Product by these unauthorized parties will void this warranty. This limited warranty does not cover, and Genesis shall

not be liable for accessories or parts not manufactured by Genesis and Genesis shall not be liable for products that are not properly cleaned and maintained as required or recommended by Genesis in the AquaSept™/AquaFit™ installation and/or operation manual. This limited warranty shall not apply to customer use outside of the Manufacturer’s recommended use and specifically this warranty shall not apply if the customer exceeds the recommended maximum use of up to 40 PSI. The limited warranty does not cover defects due to accidents or causes arising after title has passed to the Buyer.

EXCLUSIVE REMEDY; DISCLAIMER

GENESIS’ ONLY OBLIGATION UNDER THIS LIMITED WARRANTY IS THE REPAIR OR REPLACEMENT OF DEFECTIVE PARTS. GENESIS SHALL NOT BE LIABLE FOR AND HEREBY DISCLAIMS ANY DIRECT, SPECIAL, INDIRECT, INCIDENTAL, EXEMPLARY OR CONSEQUENTIAL DAMAGES OR DELAYS, INCLUDING BUT NOT LIMITED TO, DAMAGES FOR LOSS OF PROFITS OR INCOME, LOSS OF USE, DOWNTIME, COVER AND EMPLOYEE OR INDEPENDENT CONTRACTOR WAGES, PAYMENTS AND BENEFITS. THIS LIMITED WARRANTY IS GENESIS’ ONLY WARRANTY FOR THE AQUASEPT™/AQUAFIT™ PRODUCTS AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED. GENESIS MAKES NO IMPLIED WARRANTIES OF ANY KIND INCLUDING ANY IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.

No agent, distributor, dealer or other party is authorized to make any warranty on behalf of Genesis related to AquaSept™/AquaFit™ products or to assume any other liability with respect to its products.

STATUTE OF LIMITATIONS; CAP ON DAMAGES

No action may be brought against Genesis for breach of this limited warranty for the AquaSept™/AquaFit™ products or for any claim arising out of or relating to the products, more than ninety (90) days following expiration of the limited warranty period. Genesis’ total liability to the purchaser shall not exceed the original product price paid by the purchaser to Genesis.

20-70-112-INS C




Genesis Dental Technologies
200 Main Street
Elmwood, WI 54740



888.539.3907 | info@aquasept.com | www.aquasept.com

©2020 Genesis Dental Technologies. Patents Pending.

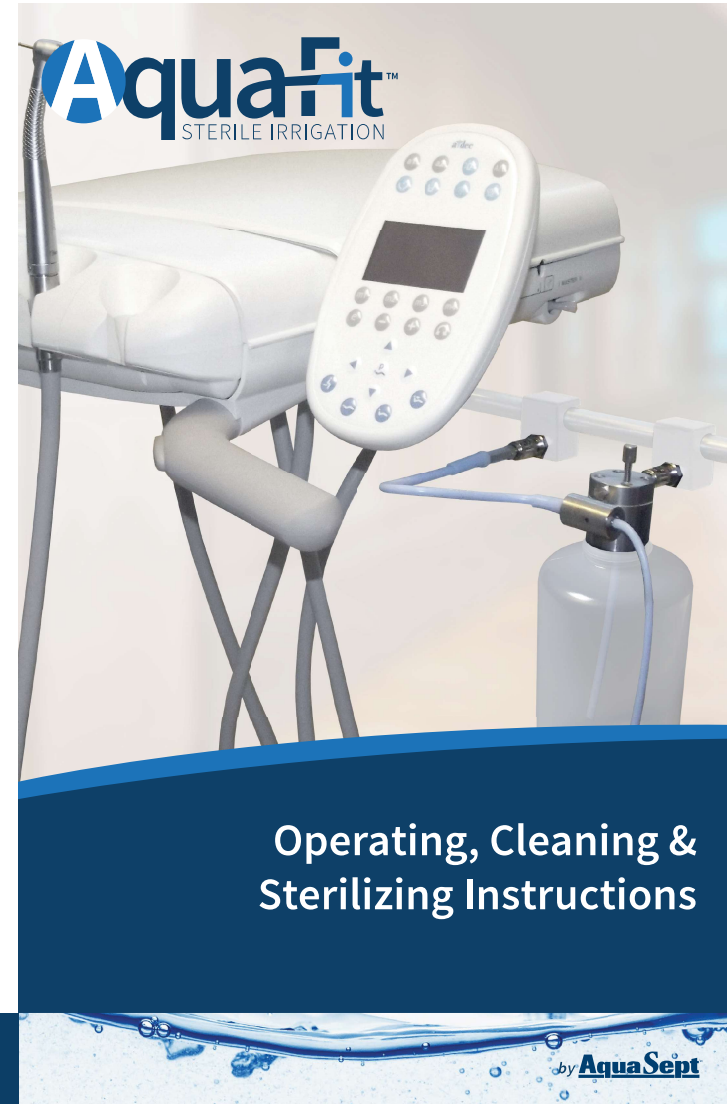

Operating, Cleaning & Sterilizing Instructions

by AquaSept

Operating Instructions for AquaFit™ System

Prior to Initial Use:

1. Unpack tubing and delivery system components.
2. The unit is delivered non-sterile. Follow steps 6 through 9 for “Routine Operation - Cleaning & Sterilizing Instructions”.
3. Proceed to “Routine Operation” instructions.

Routine Operation:

1. Fill the bottle with sterile water. **Do not use saline solution.** Do not fill the neck of the bottle.
2. Screw the bottle into Control Head. Ensure the Dip Tube is not pinched and is below the fluid level in the bottle.
3. Insert the bottle Quick Connect fitting to the dental unit mount labeled “Bottle” and the toggle Quick Connect fitting to the dental unit mount labeled “Toggle” until both click.
4. Insert the Irrigation Tubing onto the handpiece, IV port or sterile adapter**.
5. The system is now ready to use.
6. To remove unit, follow “Routine Operation - Cleaning & Sterilizing Instructions”.

**IV port / sterile adapter sold separately from handpiece manufacturer.

NOTE: Do not use sterile pour bottles sourced from other suppliers. Only 250ml (8 oz.) bottles supplied by AquaSept™ can be used.

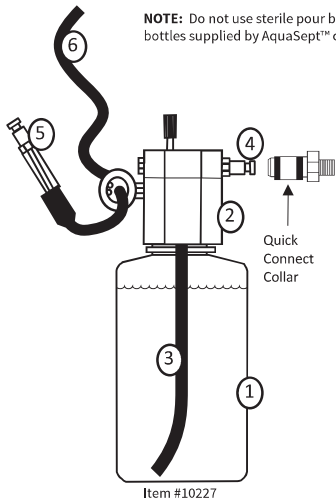


Image Key

- ① Bottle
- ② Control Head
- ③ Dip Tube
- ④ Bottle Quick Connect
- ⑤ Toggle Quick Connect
- ⑥ Irrigation Tubing

Operating Instructions for AquaFit™ System with Air / Water Syringe

Prior to Initial Use:

1. Unpack tubing and delivery system components.
2. The unit is delivered non-sterile. Follow steps 6 through 9 for “Routine Operation - Cleaning & Sterilizing Instructions”.
3. Proceed to “Routine Operation” instructions.

Routine Operation:

1. Fill the bottle with sterile water. **Do not use saline solution.** Do not fill the neck of the bottle.
2. Screw the bottle into Control Head. Ensure the Dip Tube is not pinched and is below the fluid level in the bottle.
3. Insert the bottle Quick Connect fitting to the dental unit mount labeled “Bottle” and the toggle Quick Connect fitting to the dental unit mount labeled “Toggle” until both click.
4. Insert the Irrigation Tubing onto the handpiece, IV port or sterile adapter**.
5. Place Air / Water Syringe in tool holder***.
6. The system is now ready to use.
7. To remove unit, follow “Routine Operation - Cleaning & Sterilizing Instructions”.


**IV port / sterile adapter sold separately from handpiece manufacturer.

*** Air / Water Syringe tool holder sold separately.

NOTE: Do not use sterile pour bottles sourced from other suppliers. Only 250ml (8 oz.) bottles supplied by AquaSept™ can be used.

Image Key

- ① Bottle
- ② Control Head
- ③ Dip Tube
- ④ Bottle Quick Connect
- ⑤ Toggle Quick Connect
- ⑥ Irrigation Tubing
- ⑦ Air / Water Syringe

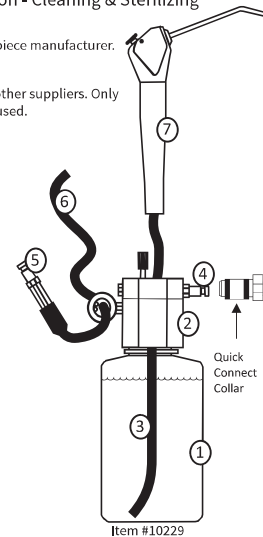

Cleaning & Sterilizing Instructions

The recommended procedure for cleaning and sterilizing the AquaFit™ units and components, including Air / Water Syringe, Control Heads, Tubing and Bottles, is described below.

Prior to Initial Use - Cleaning & Sterilizing Instructions:

The unit is delivered non-sterile. Prior to first use, follow steps 6 through 9 for “Routine Operation - Cleaning & Sterilizing Instructions.”

Routine Operation - Cleaning & Sterilizing Instructions:

1. Following use, with unit still attached to dental unit, flush water through line(s) for a minimum of 10 seconds.
 2. Disconnect unit by pushing the Quick Connect collar towards the dental unit to release. Unscrew bottle from Control Head and empty water from bottle. Replace bottle and reconnect unit. Flush the Air / Water Syringe and/or AquaFit™ unit for a minimum of 10 seconds to purge water from line(s).
 3. Unscrew bottle from Control Head.
 4. Manually clean external surface of unit and components with mild detergent (non-chlorine based). A soft brush may be used to remove any particles or debris. Rinse with water to remove any detergent.
 5. Towel dry all components.
 6. Place unit and components in one of the following:
 - Cassette wrapped with FDA cleared CSR wrap
 - Sterilization pouch
 7. **Autoclave parameters:**
 - Gravity: 132°C for 15 minutes with a 30 minute dry cycle
 - Prevacuum: 132°C for 4 minutes with a 30 minute dry cycle
- NOTE:** Do not allow components to touch the sides of the autoclave.
8. Allow unit and components to complete the dry cycle in autoclave.
 9. Allow unit and components to cool to the touch before handling.

IMPORTANT NOTICE:

- Do not exceed the autoclave manufacturer’s guidelines for load capacity.
- Do not use any chemical solutions, chlorine-based cleaners or saline solution with AquaFit™ units.
- Ensure autoclave is properly cleaned, maintained and calibrated.
- Air Quality: Dry and filtered compressed air, CO₂, N₂ or medical grade air are recommended to ensure the long working life of unit. Maintain the quality of compressed air by performing regular maintenance of the compressor according to compressor manufacturer’s recommendations.

AquaFit™ units are intended for professional use only by a licensed dental practitioner for use in general dentistry work.